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HEINE otoscopes

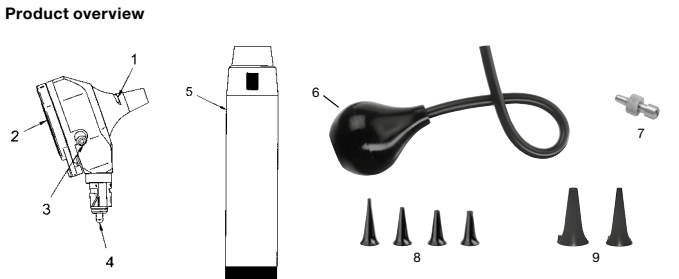
These instructions apply to the following products of the HEINE Otoscope series: BETA 200 LED, BETA 400 LED, K180 LED, mini 3000 F.O. LED, mini 3000 LED.

Please read and follow these instructions for use and keep them for future reference.

⚠️ For U.S. only:
Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information
 ⚠️ **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).
 ⚠️ **NOTE!** This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Intended Use
 HEINE otoscopes are intended for the examination of the ear, the external auditory canal and the tympanum. The instruments have optical magnifying equipment and battery or rechargeable battery powered illumination equipment. The device may only be used by qualified medical personnel and in professional healthcare facilities. Patients can be examined as often as necessary without any further restrictions. The devices are intended for transient examination. They can also be used for instrumentation with non-active medical devices (with the exception of the mini 3000 LED).

- 1 Slit
- 2 Viewing window
- 3 Insufflation port
- 4 Connector plug
- 5 HEINE Handle
- 6 Insufflation bulb
- 7 Insufflation port connector
- 8 HEINE SANALON S tips (re-use)
- 9 HEINE AllSpec tips (single use)

Setting up
 To set up the HEINE devices, screw the instrument head into the HEINE battery handle or plug it on the HEINE rechargeable handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE 2.5 V bulb
 only for use with the HEINE BETA battery handle.

Red ring = HEINE 3.5 V XHL bulb
 Black ring = HEINE 3.5 V LED bulb
 only for use with the HEINE BETA rechargeable handle and HEINE wall transformer.

This does not apply to the otoscope of the mini3000 series. Their lamp voltage is always 2.5 V independently of the handle.

Operation
 The HEINE otoscope may only be inserted into the auditory canal when a tip is fitted. Place the HEINE tip on the otoscope so that the projection inside the tip fits into the bayonet slit (1). Tighten the tip by twisting it slightly to the right.
 The magnifying lens is inside the viewing window (2). To facilitate instrumentation, the viewing window can be swivelled to both sides or folded up.

For instrumentation, use an instrument that is suitable in design and size for the otoscope being used.

The otoscopes have a connection port (3) for the optionally available insufflation bulb (6, 7) (with the exception of the mini 3000 LED).

The test of tympanic mobility can be carried out when the viewing window is closed using the insufflation bulb. Apply the pressure carefully with the insufflation bulb.

If, after switching on the unit, a sharp drop in brightness or a flashing of the light is noticed, new batteries should be inserted or rechargeable batteries recharged.

Application duration
 The otoscopes are intended for transient examination < 1 min with a break between applications of 10 minutes.
 The maximum temperature at the application point at an ambient temperature of 35 °C and in continuous operation is 44.5 °C.

The setting up and operation of the HEINE handles are described in a separate instruction of use.

Hygienic Reprocessing
 The instruction is available:
 - at www.heine.com
 - in a paper version which you can request from the address listed

Maintenance
mini 3000 F.O. LED | mini 3000 LED
 With these otoscopes, the LED cannot be changed.

Changing the light source
 ⚠️ Verify that the lamp voltage complies with the supply voltage of the handle.
 Allow the device to cool down before changing the bulb.

BETA 200 LED | BETA 400 LED | K180 LED
 Detach the otoscope from the handle and pull the bulb out of the guide tube of the connector plug (4) holding it at the narrow collar. Insert the new lamp all the way into the lamp guide.

Service
 The device has no components serviceable by the end-user.

General Notes
 ⚠️ The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and | or patient:
 All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings
 ⚠️ Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.
 Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments). This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.
 Do not modify the device.
 Use only original HEINE parts, spare parts, accessories and power sources.
 Repairs shall only be carried out by qualified persons.
 Do not shine light of the instrument directly into the eyes.
 Check the tip before each use for rough surfaces and edges.
 Disposable tips (HEINE AllSpec | HEINE UniSpec) are for single use only.
 In order not to exceed the surface temperature of 41 °C of the application part, keep within the application duration.

Disposal
 ♻️ The product must be recycled as separated electrical and electronic devices.
 ♻️ Please observe the relevant state-specific disposal regulations.

The appendix contains following tables
 - Electromagnetic disturbances – Requirements and tests
 - Technical specification
 - Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests	
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.	
	<p>Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p> <p>The supply voltage quality should be that of a typical hospital environment.</p> <p>Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.</p>
Statement for the operational environments	None
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the considered device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Conducted emissions (EN 55011/CISPR 11)*	
Radiated emissions (EN 55011/CISPR 11)	
Harmonic current emissions (IEC 61000-3-2)*	Passed*
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)*	
Immunity	See attached immunity test levels

*n |a: „Not applicable“ in the internally powered mode

Immunity test levels		
Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact discharge: ±8 kV <p>Air discharge: ±2 kV, ±4 kV, ±8 kV, ±15 kV</p>	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m <p>80 –2700 MHz <p>80 % AM at 1kHz</p></p>	
Electrical fast transients bursts (IEC 61000-4-4)*	± 2 kV <p>100 kHz repetition frequency</p>	Otoscope with a battery handle: not applicable, because of the internally powered mode.
Surges (IEC 61000-4-5)*	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)*	3 V <p>0.15 MHz – 80 MHz <p>6 V in ISM bands between 0.15 MHz and 80 MHz <p>80 % AM at 1 kHz</p></p></p>	Otoscope with a rechargeable handle: Compliance test levels are identically to the IEC 60601-1-2 test levels
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse modulation: 18 Hz; 27 V/m <p>450 MHz, FM: ±5 Hz deviation: 1 kHz sine; 28 V/m <p>710, 745, 780 MHz; Pulse modulation: 217 Hz; 9 V/m <p>810, 870, 930 MHz; Pulse modulation: 18 Hz; 28 V/m <p>1720, 1845, 1970 MHz; Pulse modulation: 217 Hz; 28 V/m <p>2450 MHz; Pulse modulation: 217 Hz; 28 V/m; <p>5240, 5500, 5785 MHz; Pulse modulation: 217 Hz; 9 V/m</p></p></p></p></p></p>	
Power frequency magnetic fields (IEC 61000-4-8)		30 A/m; 50Hz or 60 Hz
Voltage dips (IEC 61000-4-11)*	0 % U _r ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° <p>0 % U_r; 1 cycle and 70 % U_r; <p>25 30 cycles single phase: at 0°</p></p>	Otoscope with a battery handle: not applicable, because of the internally powered mode.
Short interruptions (IEC 61000-4-11)*	0 % U _r ; 250 300 cycle	Otoscope with a rechargeable handle: Compliance test levels are identically to the IEC 60601-1-2 test levels

*n |a: „Not applicable“ in the internally powered mode

Technical specification		
Environmental conditions for operation	+10 °C to +35 °C <p>30 % to 75 % rel. humidity <p>700 hPa to 1060 hPa</p></p>	
Environmental conditions for storage	+5 °C to +45 °C <p>45 % to 80 % rel. humidity <p>500 hPa to 1060 hPa</p></p>	
Environmental conditions for transport	-20 °C to +50 °C <p>45 % to 80 % rel. humidity <p>500 hPa to 1060 hPa</p></p>	
Nominal voltage	3.0 V – 3.7 V	
Nominal current	XHL: max. 760 mA <p>LED: max. 350 mA</p>	
Protection class	Internally powered	
Device classification according to IEC 62471	Exempt	
Applied part	Type BF	

LED spare parts		
Types	2.5 V	3.5 V
BETA 400 LED <p>BETA 200 LED <p>K180 LED</p></p>	X-028.88.132	X-008.88.133

Accessories		
AllSpec Disposable Tips - 2.5 mm (1x 250 pcs)		B-000.11.153
AllSpec Disposable Tips - 2.5 mm (1x 1000 pcs)		B-000.11.128
AllSpec Disposable Tips - 2.5 mm (10x 250 pcs)		B-000.11.151
AllSpec Disposable Tips - 4.0 mm (1x 250 pcs)		B-000.11.152
AllSpec Disposable Tips - 4.0 mm (1x 1000 pcs)		B-000.11.127
AllSpec Disposable Tips - 4.0 mm (10x 250 pcs)		B-000.11.150
Tip-Adaptor		B-000.11.306
Reusable Specula - 2.2 mm		B-000.11.215
Reusable Specula - 2.8 mm		B-000.11.216
Reusable Specula - 3.5 mm		B-000.11.217
Reusable Specula - 4.5 mm		B-000.11.218
Reusable Specula - 5.5 mm		B-000.11.219
Reusable Specula - 10 mm		B-000.11.220
Reusable Tip - 2.4 mm		B-000.11.107
Reusable Tip - 3.0 mm		B-000.11.108
Reusable Tip - 4.0 mm		B-000.11.109
Reusable Tip - 5.0 mm		B-000.11.110
Reusable Tips - 2.4 3 4 5 mm		B-000.11.111
Soft Disposable Tips - 3.0 mm		B-000.11.141
Soft Disposable Tips - 5.0 mm		B-000.11.142
UniSpec Disposable Ear Specula - 2.5 mm		B-000.11.242
UniSpec Disposable Ear Specula - 4 mm		B-000.11.241

Options		
AllSpec Tip-Dispenser		B-000.11.149
UniSpec Tip-Dispencer		B-000.11.146

Erklärung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati

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The CE mark indicates that the product complies with the medical device regulation (EU) 2017/745.

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	<p>Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p> <p>The supply voltage quality should be that of a typical hospital environment.</p> <p>Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.</p>	
Performance features of the ME system that have been determined to be essential to the performance	None	
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	None	

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromgnetic immunity of this equipment and result in improper operation.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the considered device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.

Test	Compliance	
RF emissions CISPR11	Group 1 Class B	
Conducted emissions (EN 55011/CISPR 11)*		
Radiated emissions (EN 55011/CISPR 11)		
Harmonic current emissions (IEC 61000-3-2)*	Passed*	
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)*		
Immunity	See attached immunity test levels	

*n | a: „Not applicable” in the internally powered mode

Immunity test levels	Test level		
Test	IEC 60601-1-2 test levels	Compliance test levels	
Electrostatic Discharge (IEC 61000-4-2)	Contact discharge: ±8 kV <p>Air discharge: ±2 kV, ±4 kV, ±8 kV, ±15 kV</p>		
Radiated RF EM fields (IEC 61000-4-3)	3 V/m <p>80–2700 MHz 80 % AM at 1kHz</p>		
Electrical fast transients bursts (IEC 61000-4-4)*	± 2 kV <p>100 kHz repetition frequency</p>		Otoscope with a battery handle: not applicable, because of the internally powered mode.
Surges (IEC 61000-4-5)*	± 0.5 kV, ± 1 kV		
Conducted disturbances induced by RF fields (IEC 61000-4-6)*	3 V <p>0.15 MHz – 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz</p>		Otoscope with a rechargeable handle: Compliance test levels are identically to the IEC 60601-1-2 test levels
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse modulation: 18 Hz; 27 V/m <p>450 MHz, FM: ±5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse modulation: 217 Hz; 28 V/m 2450 MHz; Pulse modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse modulation: 217 Hz; 9 V/m</p>		
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50Hz or 60 Hz		
Voltage dips (IEC 61000-4-11)*	0 % U _r ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° <p>0 % U_r; 1 cycle and 70 % U_r; 25 30 cycles single phase: at 0°</p>		Otoscope with a battery handle: not applicable, because of the internally powered mode.
Short interruptions (IEC 61000-4-11)*	0 % U _r ; 250 300 cycle		Otoscope with a rechargeable handle: Compliance test levels are identically to the IEC 60601-1-2 test levels

*n | a: „Not applicable” in the internally powered mode

Technical specification		
Environmental conditions for operation	+10 °C to +35 °C <p>30 % to 75 % rel. humidity 700 hPa to 1060 hPa</p>	
Environmental conditions for storage	+5 °C to +45 °C <p>45 % to 80 % rel. humidity 500 hPa to 1060 hPa</p>	
Environmental conditions for transport	-20 °C to +50 °C <p>45 % to 80 % rel. humidity 500 hPa to 1060 hPa</p>	
Nominal voltage	3.0 V – 3.7 V	
Nominal current	XHL: max. 760 mA <p>LED: max. 350 mA</p>	
Protection class	Internally powered	
Device classification according to IEC 62471	Exempt	
Applied part	Type BF	

LED spare parts		
Types	2.5 V	3.5 V
BETA 400 LED BETA 200 LED K180 LED	X-028.88.132	X-008.88.133

Accessories		
AllSpec Disposable Tips - 2.5 mm (1x 250 pcs)		B-000.11.153
AllSpec Disposable Tips - 2.5 mm (1x 1000 pcs)		B-000.11.128
AllSpec Disposable Tips - 2.5 mm (10x 250 pcs)		B-000.11.151
AllSpec Disposable Tips - 4.0 mm (1x 250 pcs)		B-000.11.152
AllSpec Disposable Tips - 4.0 mm (1x 1000 pcs)		B-000.11.127
AllSpec Disposable Tips - 4.0 mm (10x 250 pcs)		B-000.11.150
Tip-Adaptor		B-000.11.306
Reusable Specula - 2.2 mm		B-000.11.215
Reusable Specula - 2.8 mm		B-000.11.216
Reusable Specula - 3.5 mm		B-000.11.217
Reusable Specula - 4.5 mm		B-000.11.218
Reusable Specula - 5.5 mm		B-000.11.219
Reusable Specula - 10 mm		B-000.11.220
Reusable Tip - 2.4 mm		B-000.11.107
Reusable Tip - 3.0 mm		B-000.11.108
Reusable Tip - 4.0 mm		B-000.11.109
Reusable Tip - 5.0 mm		B-000.11.110
Reusable Tips - 2.4 3 4 5 mm		B-000.11.111
Soft Disposable Tips - 3.0 mm		B-000.11.141
Soft Disposable Tips - 5.0 mm		B-000.11.142
UniSpec Disposable Ear Specula - 2.5 mm		B-000.11.242
UniSpec Disposable Ear Specula - 4 mm		B-000.11.241

Options		
AllSpec Tip-Dispenser		B-000.11.149
UniSpec Tip-Dispencer		B-000.11.146

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