Instruction for use
HEINE visionPRO
video laryngoscope

Instruction for use

HEINE visionPRO

video laryngoscope

DEUTSCH	3
ENGLISH	14
FRANÇAIS	25
ESPAÑOL	36
ITALIANO	47
SVENSKA	58
NEDERLANDS	69
DANSK	80
NORSK	91
SUOMI	102
PORTUGUÊS	113

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 Please read and follow these instructions for use and keep them for future reference.

Warnings and safety information

 **CAUTION!** This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations with moderate or severe extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice. Notes are important, but not related to hazardous situations.

Intended use of the video laryngoscope

When used in combination with the HEINE visionPRO blades, the HEINE visionPRO video laryngoscope helps with both laryngoscopy and positioning an endotracheal tube.

The live image data on the product display assists with indirect laryngoscopy.

The internally powered medical device is intended for transient use in a professional healthcare facility. The healthcare professional must have adequate intubation experience.

Product overview of the video laryngoscope

Fig. 1: HEINE visionPRO Video laryngoscope

Setting up the video laryngoscope

- Charge the video laryngoscope fully before using it for the first time.
Remove the screen protection film before first use.
- Wear gloves when using the video laryngoscope.
Insert the display unit (1) into the handle (2) and, applying slight pressure, twist the components until it audibly clicks into place.

Remove the disposable blade (6) from the packaging, slide it completely over the camera arm (5) of the handle (2) and make sure it clicks into place. Make sure that the blade is firmly and securely in place.

Switch on the device by pressing the On | Off button (7) briefly.

Adjust the angle of the display (4) as desired.

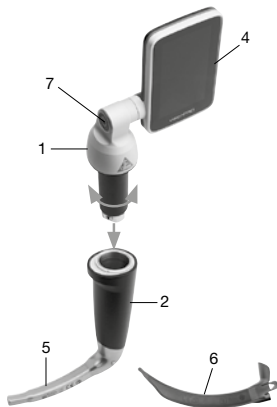
Fig. 2: Assembling the display unit and handle

Operating the video laryngoscope

- Following reprocessing, the device may only be reused when it is completely dry.
Check the blade is positioned both correctly and firmly before using the video laryngoscope.
We strongly recommend having a ready-to-use secondary system of laryngoscopes at hand.
Remove the blade from the video laryngoscope only outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see Fig. 9).
The maximum temperature at the application point at an ambient temperature of 35 °C and in continuous operation is 41.3 °C.
- If the video function fails during intubation, the device can also be used as a direct laryngoscope.
Afterwards, return the unit to HEINE Service.
visionPRO Mac 3 | visionPRO Mac 4: In the upper display section of the live image, the attached blade can be recognized. This provides an orientation.

Insert the video laryngoscope into the patient with the attached disposable blade. Use an appropriate size of blade that suits the patient or the anatomical structures.

The HyMac blade can be beneficial for Grade III or IV airways according to the Cormack-Lehane (CL) classification.

Intubate the patient. The size of the tube to be used is not restricted by the blade. Therefore, use a suitable tube size that corresponds to the patient or the anatomical structures.

After intubation, remove the video laryngoscope from the patient with the disposable blade. Switch off the device following use. Press and hold the On | Off button until the orange countdown has elapsed.

Release the fixing clip (8) and remove the disposable blade from the camera arm.

If you do not switch off the video laryngoscope after use, the device will automatically switch off after 15 minutes. A prompt with a 30-second countdown appears on the display in this regard. Press the On | Off button if you would like to interrupt the automatic switch-off process.

Should a reset be necessary, it can be reset by pressing the On | Off button for a period of 10 seconds.

Patient population

The visionPRO video laryngoscope can be used for all patients that a suitable blade is available for.

Attaching and removing the protective case

To attach the optional protective case, attach the protective case to the top front of the DS1 display. Ensure that the opening for the display hinge is correctly orientated.

Carefully pull the display protection case downwards and pull it over the display frame of the visionPRO DS1.

To remove the protective case, proceed in reverse order.

The video laryngoscope's charge level indicator

The charge level indicator is shown by a battery icon in the top part of the display. A distinction is made between the following charge level indicators:

Battery bar	Meaning	Action
Solid green 	The battery bar is green when the remaining battery capacity is between 21 % and 100 %.	No compulsory action is necessary.
Solid yellow 	The battery bar is yellow when the remaining battery capacity is between 11 % and 20 %.	The device should be charged promptly.
Solid red 	The battery bar is red when the remaining battery capacity is between 6 % and 10 %.	The device must be charged.
Flashing with a red exclamation mark 	The battery bar flashes when the remaining battery capacity is between 5 % and 0 %.	The device must be charged. Further intubation is not possible.



Troubleshooting

If the visionPRO video laryngoscope is not charging, check that the mains plug is connected to the mains and that the visionPRO video laryngoscope is correctly inserted. If necessary, reset the charging process by removing the visionPRO video laryngoscope from the visionPRO CS1 for a few seconds, unplugging the visionPRO CS1 and plugging it back in, and inserting the visionPRO video laryngoscope. If this error persists, check the electrical contacts for soiling and, if necessary, clean them carefully with a clean and lint-free cleaning cloth.

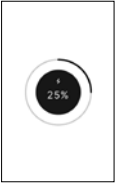
If you do not notice any improvement, please contact the manufacturer.

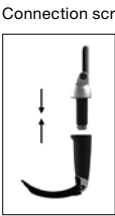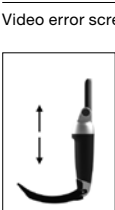Screen displays

Video laryngoscope

Screen	Meaning	Action
<p>Start screen</p> 	<p>The device switches on.</p>	–
<p>Live image</p> 	<p>The device displays the video image in real time.</p> <p>The remaining operating time is shown in minutes next to the charging indicator in the top display area.</p>	–
<p>Connection screen</p> 	<p>This animation indicates that the handle and the display unit are not connected with one another.</p>	<p>Connect the handle and the display unit.</p>
<p>Video error screen</p> 	<p>This screen shows a video error.</p>	<p>Disconnect the handle and the display unit and reconnect the two components. The live image is then displayed again.</p> <p>If this error persists, check the electrical contacts for soiling and, if necessary, clean them carefully with a clean and lint-free cleaning cloth.</p> <p>If you do not notice any improvement, take the device out of operation and contact the manufacturer.</p>

Screen	Meaning	Action
<p data-bbox="37 84 211 107">Switch-off screen</p> 	<p data-bbox="296 84 495 132">The device switches off.</p>	<p data-bbox="552 84 961 132">Press and hold the On Off button until the device switches off completely.</p>
<p data-bbox="37 387 260 435">Switch-off screen when the battery is flat</p> 	<p data-bbox="296 387 532 486">The device switches off. When the battery is low, a battery symbol is also displayed.</p>	<p data-bbox="552 387 925 460">Press and hold the On Off button until the device switches off completely. The device should be charged.</p>
<p data-bbox="37 729 242 777">Automatic switch-off screen</p> 	<p data-bbox="296 729 519 802">The device switches off automatically after 15 minutes if not used.</p>	<p data-bbox="552 729 976 802">Briefly press the On Off button if you would like to interrupt the automatic switch-off process.</p>
<p data-bbox="37 1049 260 1097">Switch-off screen when the battery is flat</p> 	<p data-bbox="296 1049 529 1097">Switch-off screen when the battery is low.</p>	<p data-bbox="552 1049 956 1097">The device must be charged before it can be used again.</p>

Screen	Meaning	Action
Switch-on screen when the battery is flat 	The battery is flat and the device cannot be switched on.	The device must be charged before it can be used again.
Charging screen 	The device displays the charge level.	–

Intended use of the charging station

The HEINE visionPRO CS1 is a charging station designed exclusively for charging the internal power supply of the visionPRO video laryngoscope.

The product must only be used by qualified medical professionals and in medical healthcare facilities.

Product overview of the charging station

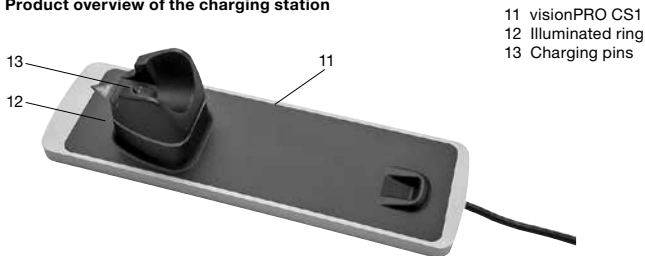
Fig. 3: visionPRO CS1 Charging station

Mounting the charging station

- ⚠ To ensure all-pole disconnection from the mains at all times, the ME device must be installed so that the switch mode power supply is accessible and disconnectable.
- 🔌 Disconnect the mains plug from the mains when assembling or disassembling the ME device.

The visionPRO CS1 is designed for the following mounting options:

- Setup on a table or a similar stable surface (Fig. 4)
- Direct wall mounting (Fig. 5)
- Wall mounting using a specific adapter on standard rails

Fig. 4:
visionPRO CS1 with an inserted
visionPRO video laryngoscope
in the table position

Fig. 5:
visionPRO CS1 with an inserted
visionPRO video laryngoscope
in the wall position

Cable routing

Loosen the screws for fastening the identification plate and route the cable of the power supply unit according to the desired fastening option (Fig. 6). Tighten the screws again after successfully aligning the cable.

Fig. 6:
Rear of the visionPRO CS1
with the various cable routing
options

Setting up the charging station

Primary plug

The primary country adapter of the power supply is interchangeable, this enables worldwide operation. Fig. 7 and Fig. 8 describe the procedure of changing the plug.

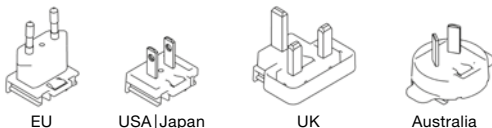
Fig. 7

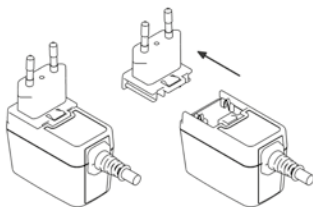
Fig. 8

⚠ After you have checked the mains voltage indicated on the nameplate of the device, connect the mains supply plug to the mains supply.

Set up or mount the visionPRO CS1 in such a way that it is easy to disconnect it from the mains. The green LED on the mains plug indicates that the power supply unit is ready for operation. To put the power supply out of operation just disconnect the power supply from the mains supply. The visionPRO CS1 is intended for setting up and use in medical areas outside the patient environment (at least 1.5 meters from the patient or patient support pursuant to IEC 60601-1, see fig. 9).

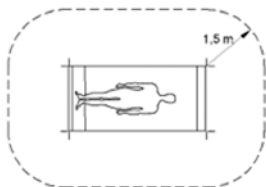
Fig. 9

Operating the charging station

To charge, place or hang the visionPRO video laryngoscope or display unit DS 1 in the visionPRO CS1 as shown in Fig. 4 or 5. The blue illuminated ring (11) on the visionPRO CS1 (10) indicates that the visionPRO video laryngoscope is inserted or hooked in and the charging station's output is switched on. To ensure that the device is being charged, the illuminated ring on the On|Off button (7) of the visionPRO video laryngoscope must light up blue in addition to the charging station, and the display (4) must briefly show the charging state.

 If the HEINE visionPRO is used with the protective case for the visionPRO DS1 display, the display has to be orientated upwards for charging in the visionPRO CS1 charging station. Check that the device is inserted correctly by determining whether the illuminated ring on the charging station is illuminated blue.

Please check the display of the video laryngoscope during the charging process to see if the device is charging.

During the charging process, the illuminated ring on the On|Off button of the visionPRO video laryngoscope flashes blue. The illuminated ring on the On|Off button lights up solid blue as soon as the visionPRO video laryngoscope is fully charged. The charge level in percent is shown for a few seconds on the visionPRO video laryngoscope's display when:

- the visionPRO video laryngoscope is inserted or hooked into the visionPRO CS1
- the On|Off button of the visionPRO video laryngoscope is pressed while it is inserted in the visionPRO CS1
- the visionPRO video laryngoscope is removed from the visionPRO CS1.

Hygienic reprocessing

The instruction is available:

- online at www.heine.com
- in a paper version which you can request from the address listed

After each use, carry out hygienic reprocessing.

 The blade is a disposable product. It must not be reused under any circumstances as serious mechanical damage may occur.

The blade is not allowed to be sterilised.

If the HEINE visionPRO is used with the protective case for the visionPRO DS1 display, this must be removed before hygienic reprocessing. The protective case must also be hygienically reprocessed after each use.

The device is not allowed to be automated reprocessed or steam sterilised.

Maintenance

There is no regular maintenance required.

Service

The device has no components serviceable by the end-user.

General Notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The protective case is a consumable article with a limited expected life cycle. Check regularly to ensure that the product is still in a safe and proper condition. If not, replace the protective case immediately.

The expected life cycle of the video laryngoscope amounts with designated use and the observation of warning and safety information up to 5 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

The expected life cycle of the charger amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

The disposable blade has a limited shelf life. Check this on the disposable blade packaging.

Store the blade in a cool and dry place. Protect from direct sunlight and temperatures >50 °C.

Charge your device during longer storage periods to protect the battery from discharge or store the battery and the device separate from each other.

Let the device accommodate to ambient conditions when it is cold.

The device may steam up if the environment is very warm or the device cools down.

If the device is steamed up, wait until the device is ready for use or wipe the affected areas.

Heating during operation is normal and harmless.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

General Warnings

 Only trained personnel may use the video laryngoscope for intubation.

Check that the device is working properly before each use. In particular, check the illumination and the live image. Do not use the device if you notice moisture in it or damage.

Do not use the device in fire- or explosive risk area.

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not use the device including its accessories outdoors.

Please ensure that a HEINE disposable blade is on the handle before using the video laryngoscope.

Do not use the device for transillumination of tissue.

Do not shine directly in the eyes to avoid dazzle, especially if the pupils are dilated.

Do not use the laryngoscope for eye reflex testing.

Check the disposable blade for sharp corners, edges and rough surfaces before use.

Switch off the device following use.

Do not short-circuit the device!

Never use a power supply with damaged housing or cable.

All load-bearing elements of the device are to be loaded only in the manner intended.

Dropping the unit or mounting accessories can cause injury and damage to property.

Check at regular intervals whether the wall mounting and the mounting accessories are in proper condition. In particular, make sure that all screws are firmly tightened.

Danger of fatal electric shock!

Use power supply in dry environment only. Avoid splashing water. Do not immerse in liquids.

Never modify a power supply or open its housing.

Connect the power supply to a regular outlet only.

Protect connectors from moisture.

Never carry the power supply by its electrical cable or pull at the cable.

For separating the connection always pull on the plug, never on the cable.

Cables must be installed safely to avoid tripping over.

Disposal

Carry out reprocessing of the video laryngoscope before disposal.

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
	The supply voltage quality should be that of a typical hospital environment.
	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
	NOTE: The EMISSIONS characteristics of this equipment (visionPRO) make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or reorienting the equipment.
Performance features of the ME system that have been determined to be essential to the performance	None
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used. The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.
Test	Compliance
RF emissions CISPR 11	Group 1 Class B (charging station) Class A (video laryngoscope)
Conducted emissions (EN 55011/CISPR 11)	Passed
Radiated emissions (EN 55011/CISPR 11)	
Harmonic current emissions (IEC 61000-3-2)	
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)	
Immunity	See attached immunity test levels

Immunity test levels

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ± 8 kV Air Discharge: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80–2700 MHz 80 % AM at 1 kHz	
Electrical fast transients bursts (IEC 61000-4-4)	± 2 kV 100 kHz repetition frequency	
Surges (IEC 61000-4-5)	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)	3 V 0.15 MHz – 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: ± 5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50 Hz or 60 Hz	30 A/m; 60 Hz
Voltage dips (IEC 61000-4-11)	0 % U_T ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U_T ; 1 cycle and 70 % U_T ; 25 30 cycles Single phase: at 0°	
Short interruptions (IEC 61000-4-11)	0 % U_T ; 25 300 cycles	

Technical specification

Environmental conditions for operation	+5 °C to +35 °C 10 % to 80 % non-condensing 700 hPa to 1060 hPa
Environmental conditions for storage	-20 °C to +50 °C 0 % to 95 % non-condensing 700 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 0 % to 95 % non-condensing 500 hPa to 1060 hPa
Protection class	Handle: IP68 Display Unit: IP6X
Power supply (video laryngoscope)	Internally powered (3,7 dc, 2.300 mAh)
Power supply (charging station)	100 – 240 Vac; 50–60 Hz
Power consumption (charging station)	300 mA – 150 mA

Accessories (List of compatible blades)

Article	Article number	Overall length	Blade length	Distal width	Material
visionPRO Mac 3 (10x)	F-000.22.243	135 mm	117 mm	14,5 mm	ECO ABS
visionPRO Mac 3 (50x)	F-000.22.253				
visionPRO Mac 4 (10x)	F-000.22.244	151 mm	132 mm	15,5 mm	
visionPRO Mac 4 (50x)	F-000.22.254				
visionPRO HyMac 3 (10X)	F-000.22.246	134 mm	115mm	15 mm	
visionPRO HyMac 3 (50X)	F-000.22.256				
visionPRO Mac2 (10X)	F-000.22.242	119 mm	101 mm	14,5 mm	
visionPRO Mac2 (50X)	F-000.22.252				

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Do not use if the packaging is damaged, and follow the instructions for use. If the packaging is damaged, the blade may be contaminated. If the packaging is damaged, the video laryngoscope, the blade or the charging station may be mechanically damaged.

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