

## Important preparation information

- Before performing tonometer measurements a corneal topical anesthetic is required
- A fresh AccuTip should be applied for each patient to protect the eye and instrument tip as well as prevent cross-contamination
- Failure to apply a tip cover prior to taking a reading may result in a corneal abrasion
- Prior to use, examine each AccuTip cover for yellowing, cracks or a sticky texture. The latex material used in the AccuTip covers can degrade which may result in leakage of wetting solution and instrument damage.
- If the "Use before" date stamped on the box is expired do not use. Usage before this date helps to guard against the possibility of shelf wear.

**Warning: This product contains latex which may cause allergic reactions**

## Application

Place cardboard sleeve over the tip of the tonometer and slide the cover down until it sits in the groove.



## Placement

Make sure that the rubber cover is flat across the tip and not too taut or too loose. This could cause issues with readings.



## Symbols

LOT

### Lot number

- Numero do lot / Numero di lotto / Ch-B. / Numero de lote



### See insert

- Voir notice a l'interieur / Vedere insert / Siehe beilage / Veá la hoja intercalada



### Do not reuse

- Ne pas reutiliser / Non riutilizzare / Nicht wiederverwenden / No reusar



### Use by date

- Utiliser avant le (date) / Scadenza / Verfallsdatum / Use antes de la fecha

REF

### Catalog number reference

- Numero di catalogo / Bestellnummer / Numero de catalogo

## Storage

AccuTip covers should be stored in a dry area that is between 35° and 80° Fahrenheit (2° - 27° Celsius).

## AccuTip covers

Single use AccuTip covers not only help prevent the transfer of infection but also protect the tip of the tonometer.

AccuTip covers are compatible with both the Keeler AccuPen and Reichert Tono-Pen.



FR

### Couvre-pointe AccuTips

Informations et instructions concernant le produit

IT

### Copertura per la punta AccuTips

Informazioni sul prodotto ed istruzioni per l'uso

DE

### AccuTips spitzenhülle

Produktinformation und gebrauchsanweisung

ES

### La cubierta de punta AccuTips

Instrucciones e informacion sobre el producto



## Contact us

[keeler.co.uk](http://keeler.co.uk)  
[keelerusa.com](http://keelerusa.com)

**UK:** +44 (0) 1753-857177

**USA:** +1 (800) 523-5620

**India:** +91 (99) 3031-1090

**Brazil:** +55 (11) 4302-6053

**China:** +86-18512119109

  
A Halma company



 Keeler • 3222 Phoenixville Pike, Bldg. 50, Malvern, PA 19355

AX9919R-EU REV D

Effective date: March 7, 2023



## AccuTip covers

AccuPen tip cover information  
and instructions